
Award Winning
Circulation Therapy

HORSEWARE®
ICE-VIBE®
CIRCULATION THERAPY

USER MANUAL
Manuel de l'Utilisateur | Gebrauchsanweisung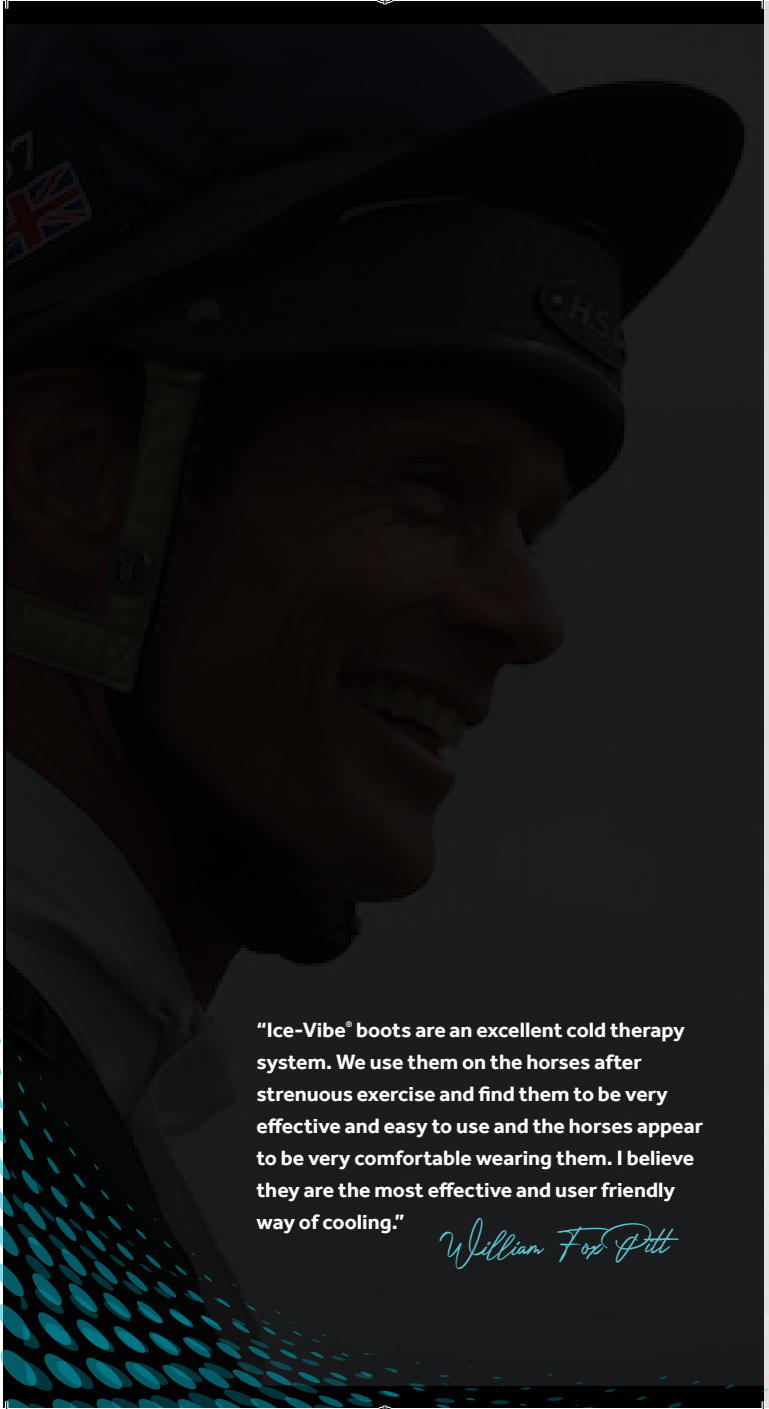

"Ice-Vibe® boots are an excellent cold therapy system. We use them on the horses after strenuous exercise and find them to be very effective and easy to use and the horses appear to be very comfortable wearing them. I believe they are the most effective and user friendly way of cooling."

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Each ICE-VIBE® Pack or Knee Pack contains:

- ✓ 2 boots or wraps
 - ✓ 2 cold packs
 - ✓ 2 integrated vibrating panels
 - ✓ 1 battery charger and 4 universal adapters
 - ✓ Spare parts sold separately
- 
- 

Each Hock Wrap Pack Contains:

- ✓ 2 wraps
 - ✓ 4 cold packs
 - ✓ 2 integrated vibrating panels
 - ✓ 1 battery charger and 4 universal adapters
 - ✓ Spare parts sold separately
- 

Pack contents

ICE-VIBE®
BOOTS

ICE-VIBE®
XFULL BOOT

ICE-VIBE®
KNEE WRAP

ICE-VIBE®
HOCK WRAP

Accessories

ICE-VIBE®
COLD PACKS
(SOLD AS A PAIR)

ICE-VIBE® HOCK
COLD PACKS
(SOLD AS A PAIR)

ICE-VIBE® KNEE
ICE PACKS

COOL BAG
(NOT SOLD SEPARATELY)

INTEGRATED VIBRATION
PANELS & BATTERY

CABLE & TRANSFORMER

About your Ice-Vibe® Boots

How do they work

Ice-Vibe® boots are rechargeable vibrating boots that boost circulation in horse's legs. They combine the effects of cooling and massage therapy to effectively treat strains, ligament and tendon damage. The boots can be used with and without cold packs to reduce swelling and stimulate blood flow. Cooling slows down blood flow to prevent swelling, while the massage effect encourages lymph to move, draining inflammation and stimulating repair to damaged tissue. Using the boots before exercise can help reduce the risk of injury as increasing blood flow to the tendons and ligaments makes them more elastic. Everyday wear and tear on joints can be managed with daily use of the boots, and rehabilitation through injury can be achieved more quickly.

Setting up your boots

Insert vibrating panels into pockets flat side to the horse's leg. Use one panel in each boot. The battery/control unit is marked with an on/off switch and is integrated into the panel.

Place cold packs in fridge/freezer BEFORE USING.

Using your Ice-Vibe® boot

Place the cold packs on your horse's legs. Apply to the lower leg area and wrap ensuring the Velcro aligns symmetrically with its opposing counterpart. The packs are clearly marked left and right. Place the boot over the top of the cold pack and secure with velcro straps. **Switch the boot on by holding down the on/off button for 3 seconds at the back of the boot where clearly marked on/off.**

Your boot has three settings. To change the setting press the on/off button once. Your boot will switch off after ten minutes on setting one and twenty minutes setting two and three. If you need to switch off your boot manually hold the on/off button down for 3 seconds. If your horse has white socks or sensitive skin or you are using the pack during the winter months please leave cold packs out of the freezer for 5 minutes before using. **If you have any queries on how to use of boots please contact icevibe@horseware.com**

Setting up your boots

Insert vibrating panels into pockets flat side to the horses leg. Use one panel in each boot. The battery/control unit is marked with an on/off switch and is integrated into the panel. **Place cold packs in fridge/freezer before using.**

Using your boots with cold packs

Your cold packs can be cooled in a fridge or freezer for a minimum of 2 hours before use. Fridge may be adequate for day to day use. Always use your cold packs if swelling is present.

Using your boots with warm packs

Horses suffering with joint stiffness may benefit from warm massage. You can heat the packs by soaking them in warm water. Do not microwave them. Do not soak in boiling water as you may burn your horse. Always start your horse on level one, once a day for two to three days to allow him to adjust to the boots. **Setting two is your main setting before and after exercise.**

How to charge the battery

1. The battery is integrated into the top of the panel. It has been partially charged, but you may need to recharge it before you can switch on your device for the first time.
2. Connect the charger to a wall outlet.
3. Connect the charger to the battery. The battery is integrated into the panel; the connection is at the bottom of the panel.
4. Battery will fully charge in two hours with one battery or in four hours for two batteries simultaneously charging.
5. Disconnect the charger from the wall outlet when the charger is not in use. Your device can be charged from a USB port e.g. A laptop, when a wall outlet is not available. Should you require a second USB charger the Ice-Vibe battery requires a USB Type-C size cable.
6. If you're going to store your boots for more than a month, store with battery fully charged. If stored and not charged the lifespan of your battery will be reduced.

*** If your battery is not integrated, remove battery, connect charger and charge both batteries for 4 hours.**

Your boots should last for two hours once charged

Always start your horse on the first level that switches off after ten minutes for the first three days until his legs get used to the Ice-Vibe® boots.

What settings to use

Setting 1:

Always start on setting one for the first few days to allow your horse's legs to adjust to the wrap and to make sure the leg does not swell from use

Use for first few days of rehabilitation

Setting 2:

Use to prepare for exercise without the cold packs

Use after exercise with cold packs

Use for rehabilitation

Setting 3:

This is the strongest setting

Should only be used with cold packs

Should be used on larger boned horses or horses with old excess scar tissue


Please note:

- When swelling and bruising is present always use with cold packs
- If your horse's legs swell after use s/he may have a fracture or an infection
- If you are unsure please consult your veterinarian

Always start your horse on setting one for the first three days to allow your horses legs to adjust to the boot and to make sure the leg does not swell from use. If your horse's legs swell after use your horse may have a fracture or infection.

The Ice-Vibe® boots are designed to assist you to prepare your horses legs before exercise and to minimise soreness and bruising after exercise they are not a medical device.


BEFORE THERAPY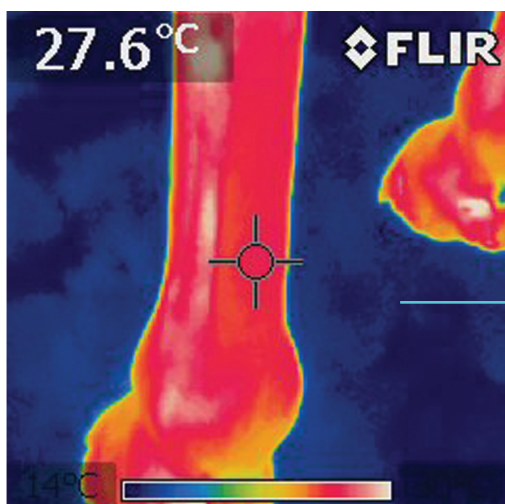

Resting normal leg
before therapy.


Thermal Photographs

AFTER USING ICE PACK & VIBRATION THERAPY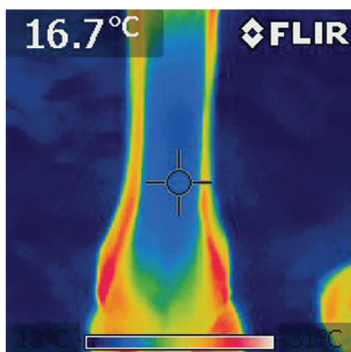

Tendon after using ice pack and vibration with Ice-Vibe® Boot. Leg is significantly cooler.

AFTER THERAPY USING ONLY VIBRATION

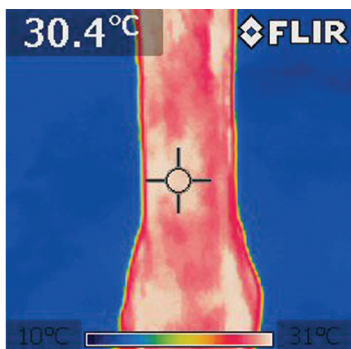Prevents Injuries:
Leg after using vibration panels only with Ice-Vibe® Boot. Leg warmed up and ready for exercise.

About your Ice-Vibe® Hock and Knee Wraps

How do they work

Ice-Vibe® hock and knee wraps help increase circulation to your horse's hocks and knees. However, they cannot be used on infection as the massage effect may spread the infection. Do not use on fractures in the first four weeks as it may irritate and cause some swelling.

Ice-Vibe® wraps are designed to help you prepare your horse's legs before exercise and reduce inflammation, discomfort and bruising after.

Remember: this is not a medical device.

Setting up your Hock / Knee Wraps

1. Insert vibrating panels into pockets with the flat side to the horse's leg and the battery end of the panel going into the wrap first. Use one panel in each wrap.

The battery/control unit is marked with an on/off switch and is integrated into the panel. When inserted into wrap you should be able to see on/off button on outside of wrap.

The hock wraps give you the option of having the panel on the inside or the outside of the hock.

3. The panel for the knee wrap goes into the middle of the wrap.
4. Place cold packs in fridge/freezer for a minimum of two hours before.

Turning on your Ice-Vibe® wraps

1. Switch the panel on by holding down the on/off button at the bottom of the wrap for 3 seconds.
 2. Your wrap has three settings. It switches on at setting one. To change the setting press the on/off button once.
 3. Your wrap will switch off after ten minutes on setting one, press the button once more for setting two, this lasts twenty minutes.
 4. Press once more for setting three. This also lasts 20 minutes.
 5. If you need to switch off your wrap manually hold the on/off button down for 3 seconds.
- 
- 

Always start your horse on level one, once a day for a couple of days to allow him to adjust to the wraps.

Setting two is the best setting for before and after exercise.

Fitting your Ice-Vibe® Knee Wrap

1. Place the wrap around your horse's knee, with the massage panel sitting at the front middle of the knee and the back of the knee coming out through the gap at the back of the boot.
2. Secure the top Velcro strap above your horse's knee first.
3. Fasten the outer elasticised strap firmly over the top of the first strap. (see picture)
4. Ensure back of the knee is exposed (see picture)
5. Secure the bottom strap below the knee loosely.
6. When treatment is finished remove the wrap.

Fitting your Ice-Vibe® Knee Wrap with cold pack

1. To attach the cold pack to the inside of the knee wrap place it inside the front of the wrap, with the velcro piece facing the outside of the wrap.
2. Then fold the velcro down over the top to secure it.

As the cold pack is applied directly onto the knee we advise leaving it out of the freezer for 5 minutes prior to use. Your cold packs can be cooled in a fridge or freezer for a minimum of 2 hours before use. Always use your cold packs if swelling is present.

Insert vibrating panel into pocket

Making sure recharge point fits to bottom of pocket enligh with ON/OFF switch on opposite side

Insert cold pack to inside of wrap

Attach velcro to outside of wrap

Fitting your Ice-Vibe® Hock Wrap

(YOUR WRAPS ARE CLEARLY MARKED LEFT AND RIGHT)

1. Place the wrap around your horse's hock, with the point of the hock sitting through the hole in the wrap.
2. Secure the top Velcro strap above your horse's hock.
3. Then secure the second strap firmly.
4. Then secure the third strap firmly.
5. Secure the bottom strap below the hock firmly.
6. When treatment is finished remove the wrap.

Fitting your Ice-Vibe® Hock Wrap with cold packs

The hock wrap comes with two cold packs that are the same size. The packs fit into the two pockets that sit either side of the hock. Cold packs can be cooled in a fridge or freezer for a minimum of 2 hours before use. Always use your cold packs if swelling is present.

Using your Ice-Vibe® Wraps with warm packs

Horses suffering with joint stiffness may benefit from warm massage. You can heat the packs by soaking them in warm water. Do not microwave them. Do not soak in boiling water as you may burn your horse.

Insert vibrating panel in pocket behind cold pack

Insert cold packs either side of Hock Wrap

When fitting wrap, close straps numerical order as shown

Hock/Knee Wraps Settings

When fitting the Ice-Vibe® Hock Wrap, close straps numerical order as shown below:

When fitting the Ice-Vibe® Knee Wrap, close straps numerical order as shown below:

Fitting Cold Packs

Each pair has a left and right cold pack Flared end to cover the Fetlock Joint

User guide safety: circulation wraps

- Only qualified personnel may repair this product.
 - Keep your device dry it is not water resistant.
 - Do not store the electronic components in extreme hot or cold temperatures.
 - Do not drop, knock or shake device.
 - Do not allow horses to chew or stand on device as it contains delicate circuit boards.
 - Use battery and charger for their intended purposes only. Improper use, or use of unapproved batteries or chargers may present a risk of explosion or other hazard and will invalidate warranty.
 - Never use a damaged battery or charger.
 - Do not place cold packs in a microwave.
 - Do not heat cold packs.
- 
- 

All electrical and electronic products and batteries must be taken to separate collection at the end of their working life. This requirement applies in the European Union. Do not dispose of these products as unsorted waste.

Warranty

The warranty period begins from the time of original purchase by the first end user. The electronics have a 6 month warranty or 1000hrs. The limited warranty period will not be extended or renewed due to subsequent resale. This limited warranty does not cover defects caused by rough handling or chewing from the horse, defects caused by sharp items or damage caused by misuse of the product. This limited warranty does not cover defects or damaged cause by the fact that the product was connected or used with any product not supplied by Horseware®. This limited warranty is not enforceable if the product has been exposed to dampness, extreme thermal or environmental conditions, or to spillage of food or liquid or chemical products. This limited warranty is not enforceable if the product has been modified or repaired by an unauthorized person. This limited warranty is not enforceable if the battery has been opened, broken or shows evidence of tampering.

To view our other Ice-Vibe® products or to make an enquiry contact us via www.horseware.com or email us at icevibe@horseware.com

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